## IFU

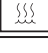
(Instruction for use)

### Bisturi Circular / Circular Scalpel / Bisturi Circular

PORTUGUÊS

ENGLISH

ESPAÑOL

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Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological concepts, so, for best results it should only be used with original associated components according to Emfils instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthodontists and Oral Maxillofacial Surgeons.

**DESCRIPTION**

The Emfils circular scalpels form a set of 6 pieces between short and long models of different diameters, according to the implant model which is covered by keratinized gum that to be exposed. The circular scalpel, at its active tip, has a sharp circular geometry, as well as a transverse hole that will facilitate the internal cleaning of the instrument. At the other tip, the circular scalpel has an anatomically designed body for bi-digital use, and has an independently-driven washer and hole for dental floss installation, in order to increase safety during use. Although the washer has circular movements independent of the body, it is fixed and can't be disassembled from the instrument.

The raw material used is tempered stainless steel, according to ASTM F 899.

The Emfils circular scalpels are presented in the table below:

Sales Code	Sharp Diameter	Size	
		Length	
		Short	Long
BCC-3310	3,5	X	
BCC-4010	4,0	X	
BCC-5010	5,0	X	
BCL-3320	3,5		X
BCL-4020	4,0		X
BCL-5020	5,0		X

**APPLICATION**

The Emfils circular scalpel is a surgical instrument used to expose the sub gingival implant, through a cut of the keratinized gum that covers it.

**CONTRAINDICATIONS**

Circular scalpels do not present contraindications, provided they are used correctly for the purposes indicated.

**HANDLING**

Once sterilized, the circular scalpel should be used in compliance with all biosafety and surgical asepsis requirements.

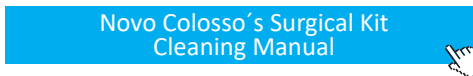
All procedures must be performed by qualified professionals and the handling of the product is inherent in the training of the professional who will be responsible for using the instruments in the best possible way, respecting the Emfils indications for each instrument.

Select the circular scalpel that matches the installed implant diameter. Also choose the scalpel that has the appropriate size to where the procedure will take place. For patient safety, thread and tie a dental floss into the hole in the washer at the top of the scalpel, giving greater safety to the procedure. Take the circular scalpel to the point where the implant is installed and perform circular movements on the gingival tissue until it detaches and exposes the implant cap.

**SANITATION AND STERILIZATION**

The circular scalpels are reusable, provided non-sterile and should be sanitized and sterilized before and after use by following the steps

mentioned in the link below:

**PRESENTATION**

The Emfils circular scalpels are individually packaged in tyvek® surgical paper and transparent film display. The instrument undergoes a decontamination process that eliminates any type of dirtiness, but the sanitation process, described above, must be followed as soon as it's first used.

**PRECAUTIONS**

- Do not use if the product packaging is broken;
- This product should be used sterile. Do not sterilize under dry heat (stove). Always use steam and pressure autoclave;
- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- Should be careful in handling and the sanitation process with the scalpel because it is a sharp metal instrument, avoiding any type of physical trauma and loss of cut;
- Be sure to use the diameter of the circular scalpel compatible with the installed implant diameter;
- Circular scalpels should not be re-sharped;
- The non-replacement of a scraped scalpel, as recommended by Emfils, may not properly perform cutting of gingival tissue, compromising the success of the procedure;

- Due to the reduced opening of the mouth in the posterior region, the use of the instrument may be difficult;
- Maximum care is recommended when using the instruments so that they are not swallowed or aspirated by the patient. Always use the dental floss technique on the washer hole at the top of the scalpel, giving greater safety to the procedure;
- Before each procedure, check the conditions of the Emfils instruments, always respecting their useful life. Replace the instruments in case of damage, erasure, deformation and excessive wear.
- The use of excessive force may compromise the procedure;
- Always use the sequence of Emfils products. The use of instruments and / or components from other manufacturers and systems does not ensure the perfect function and exempts Emfils from offering any product warranty.

**ADVERSE EFFECTS**

When properly indicated, where the professional identifies the position of the implant in the buccal cavity, and the patient has a keratinized gum in an adequate quantity for the procedure, Emfils circular scalpel has no adverse effects.

**STORAGE CONDITIONS**

The product should be stored in a dry place and in the original packaging at room temperature and not expose to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

**DISPOSAL OF MATERIAL**

All consumption products and materials used in the surgery for the insertion of dental implants and/or prosthetic components can endanger the health of those who handle them, after use. Before disposing them into the environment, we recommend to refer to and comply with the effective legislation.

**LIFE CYCLE**

This product is recommended for up to 30 uses, provided that Emfils-recommended conditions of use are followed. Regardless of the number of times the instrument has been used, the professional must always evaluate the instrument's conditions after each use.

**SHELF LIFE**

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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