IFU

(Instruction for use)

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Favor contactar seu distribuidor autorizado.

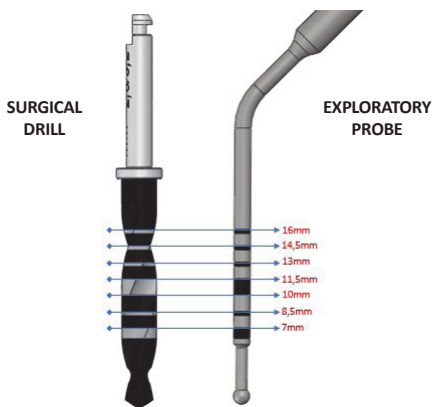
This product has been designed to integrate clinical, mechanical and biological concepts, so for best results it should only be used with original associated components according to Emfils instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthetists and Oral Maxillofacial Surgeons.

DESCRIPTION

The Emfils exploratory probe has a cylindrical handle and the 45 ° angled end with laser markings on the length of EMFILLS implants.

The raw material used in the Sonda Exploradora is stainless steel surgical, according to ASTM F899.

APPLICATION

This product is an instrument used in dental surgeries to measure and inspect the surgical shop.

It should not be used on straight mini pillars, only on angled ones.

CONTRAINDICATIONS

No contraindications, provided it is used correctly for the indicated purposes.

HANDLING

During and after perforation of the surgical shop, the exploratory probe is used to detect possible trepanations of the bone plates of different anatomical structures, as well as to measure the depth of the perforation and consequently the selection of the length of the Implant to be used. It has at one end a spherical tip with 1.5mm in diameter that facilitates the exploration and drilling of drilling.

SANITATION AND STERILIZATION

The Emfils exploratory probe is reusable, supplied non-sterile and must be sanitized and sterilized before and after use, following the recommendations of the document available at the link below:

[Novo Colosso's Surgical Kit Cleaning Manual](#)

PRESENTATION

The Emfils exploratory probe is individually packaged in tyvek® type surgical paper packaging and clear film

display. The instruments undergo a decontamination process that eliminates any type of dirt, but the hygiene process, described above, must be followed as soon as it is first used.

PRECAUTIONS

- Do not use if the packaging has been tampered with;
- This product should be sterile. Do not sterilize under dry heat (greenhouse), always use steam autoclave and pressure;
- Chemical sterilization is not recommended;
- Do not autoclave the explorer probe in its original packaging;
- Before each procedure check the conditions of the instruments In fi ls, always respecting their useful life. Replace instruments in case of damage, missing markings, deformation and wear;
- Improper planning may compromise the performance of the implant / prosthesis assembly resulting in system failure, such as implant loss or fracture, loosening or fracture of prosthetic components and / or screws.
- Always use the product sequence In fi ls. The use of instrumentation and / or prosthetic components from other manufacturers does not guarantee the perfect function of the Implant System and exempts any warranty of the product.

ADVERSE EFFECTS

Adverse effects will only occur if the choice of instruments is inadequate, which may result in damage to the implant or component to be installed.

STORAGE CONDITIONS

The product should be stored in a dry place in its original packaging at room temperature and not exposed to direct sunlight. Incorrect storage may influence the original characteristics of the carton and the product.

DISPOSAL OF MATERIAL

The operator must always evaluate the condition of the instrumentation after each use and it must be discarded in accordance with current regulations, complying with the environmental requirements and the criterion of potentially dangerous products, and if the instrument is damaged, markings erased, deformed and worn. , taking into account the different levels of contamination, physical risks and selective collection.

LIFE CYCLE

The exploratory probe must be discarded when it loses its functionality or the user detects excessive wear.

SHELF LIFE

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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