## IFU

(Instruction for use)

### Sonda Exploradora Guide / Guide Explorer probe / Sonda Exploradora Guide

PORTUGUÊS

ENGLISH

ESPAÑOL

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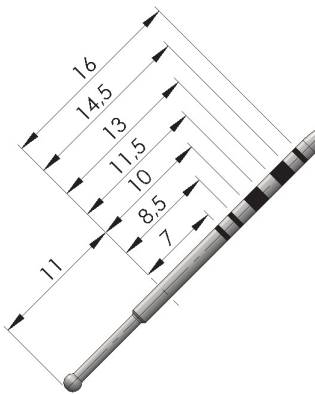
Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological beings and therefore, to obtain results, it must have been improved to the original standards according to Emfil instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthetists and Oral Maxillofacial Surgeons.

**DESCRIPTION**

The Exploratory Probe Guide (SEG-1515) has cylindrical shape providing anatomical palmar digital restraint. At the upper end has a rounded tip to avoid damaging the contact surface (bone and mucosa). It has laser engraving with the manufacturing lot for traceability and the heights corresponding to the drilling protocols of the surgical drills. It has 45 ° oblique body to facilitate insertion at any point in the oral cavity. Already has compensation of 11mm referring to the Guided Surgery concept as shown below:

The raw material used in the Sonda Exploradora is stainless steel surgical, according to ASTM F899.

**APPLICATION**

The Exploratory Probe Guide is indicated to determine bone plate thicknesses, to detect possible defects in the surgical alveolus after perforation and to measure the depth for selection of the length of the implant to be used.

**CONTRAINDICATIONS**

It is contraindicated to use the measurement function of the Scanner Guide without the use of the Surgical Guide. The Scanner Guide is unique for use in the guided surgery protocol, having an 11mm offset on the height recordings.

**HANDLING**

After the initial drilling with the drill bits and drill Ø2,5 in the planned place, insert the metal rod of the Probe into the cavity realized. Measure the implant length using the laser engraving indicated on the probe stem (7.0

mm / 8.5 mm / 10.0 mm / 11.5 mm / 13.0 mm / 14.5 mm / 16 , 0 mm). In all subsequent phases of the protocol, the user can take measurements or even check the quality of the remaining bone table. The engraving has 11mm of washer compensation and drill guide, being an exclusive item for the Guided Surgery Emfil's Guide technique.

**SANITATION AND STERILIZATION**

The Exploratory Probe Guide is supplied non-sterile and should be sanitized and sterilized before and after use, following the recommendations of the document available at the link below:

[Novo Colosso's Surgical Kit Cleaning Manual](#)

**PRESENTATION**

The Exploratory Probe Guide is individually wrapped in tyvek® surgical paper packaging and clear film display. The instruments undergo a decontamination process that eliminates any type of dirt, but the hygiene process, described above, must be followed as soon as it is first used.

**PRECAUTIONS**

- Do not use if the packaging has been tampered with;
- Attention with the use of the exploratory probe Guide (SEG-1515), because it has compensation of 11mm in the recordings of height;
- This product should be sterile. Do not sterilize under dry heat (greenhouse), always use steam autoclave and pressure;
- Chemical sterilization is not recommended;
- Do not autoclave the keys in their original packaging;
- Before each procedure check the conditions of the instruments In fi ls, always respecting their useful life. Replace instruments in case of damage, missing markings, deformation and wear;
- Improper planning may compromise the performance of the implant / prosthesis assembly resulting in system failure, such as implant loss or fracture, loosening or fracture of prosthetic components and / or screws.
- Always use the product sequence In fi ls. The use of instrumentation and / or prosthetic components from other manufacturers does not guarantee the perfect function of the Implant System and exempts any warranty of the product. Keys for installing components from other systems are indicated for use in other implant systems as recommended by the component manufacturer.

**ADVERSE EFFECTS**

Adverse effects will only occur if the choice of instruments is inadequate, which may result in damage to the implant or component to be installed.

**STORAGE CONDITIONS**

The product should be stored in a dry place in its original packaging at room temperature and not exposed to direct sunlight. Incorrect storage may influence the original characteristics of the carton and the product.

**DISPOSAL OF MATERIAL**

The operator must always evaluate the condition of the instrumentation after each use and it must be discarded in accordance with current regulations, complying with the environmental requirements and the criterion of potentially dangerous products, and if the instrument is damaged, markings erased, deformed and worn. , taking into account the different levels of contamination, physical risks and selective collection.

**LIFE CYCLE**

The Explorer probe Guide should be discarded when it loses its functionality or the user detects excessive wear.

**SHELF LIFE**

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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