## IFU

(Instruction for use)

### Chave de Instalação / Installation Driver / Llave de Instalación

PORTUGUÊS

ENGLISH

ESPAÑOL

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Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological concepts, so, for best results it should only be used with original associated components according to Emfils instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthodontists and Oral Maxillofacial Surgeons.

**DESCRIPTION**

The installation drivers for Emfils components have 4 models: hexagonal 1.2 mm, 2.0 mm, 2.1 mm and 2.4 mm, and 1 square model 2.0 mm, and all these models have 3 different sizes: short, medium and long. In the top interface, which connects to the torque wrench gear or to the counter-angle motor, all short-sized and long drivers have a 3.9 mm universal square fit and all medium drivers have counter-angle fit. In total, we have 15 driver options for use only in the Emfils implant system. The hexagonal drivers for installation of Emfils implants (2.0, 2.1 and 2.4 mm) have at their lower tip, a collection system that fixed the implant to the driver, and helps to remove it from the transparent tube of the package, without any necessary manipulation of the implant. There is also the option, if the user understands that he needs a longer driver, to use the driver extension, which increases the actuation of the medium and long drivers for counter-angle, where plus 2 options for each key model, totaling 10 more options. These drivers and options are an integral part of the Emfils surgical kit.

We also have the drivers to install components of other systems, where Emfils has composed a prosthetic kit that contains practically all models of drivers existing in the market, in the option of medium size, with counter-angle fitting, and can have its actuation length increased with the use of the driver extension. All drivers for installing components from other systems are an integral part of the Emfils prosthetic kit.

All drivers are laser marked with the size of their slot, for easy identification and storage by the user. All internal fitting drivers have a transverse hole in the side of the body for easy cleaning.

The raw material used in all of the drivers is tempered stainless steel, according to ASTM F899.

All the installation driver models are presented in the table below:

Sales Code	Installation Component							Connection Dimension	Connection Type		Length	Interface Type
	Implant						Screw		Internal	External		
	MT (CM)	ZL (LZ)	EH (HE)	IH (HI)	OBA (CUP)	OBO (CUO)						
IHC-1214							X	Hex. 1,2	X	Short	Sqr. 3,9	
IHC-2014	X							Hex. 2,0	X	Short	Sqr. 3,9	
IHC-2114			X					Hex. 2,1	X	Short	Sqr. 3,9	
IHC-2414				X				Hex. 2,4	X	Short	Sqr. 3,9	
ILCA-1226							X	Hex. 1,2	X	Long	Sqr. 3,9	
ILCA-2026	X							Hex. 2,0	X	Long	Sqr. 3,9	
ILCA-2126			X					Hex. 2,1	X	Long	Sqr. 3,9	
ILCA-2426				X				Hex. 2,4	X	Long	Sqr. 3,9	
IMCA-1221							X	Hex. 1,2	X	Medium	Counter Angle	
IMCA-2021	X							Hex. 2,0	X	Medium	Counter Angle	
IMCA-2121			X					Hex. 2,1	X	Medium	Counter Angle	
IMCA-2421				X				Hex. 2,4	X	Medium	Counter Angle	
IQC-2014					X	X		Sqr. 2,0	X	Short	Sqr. 3,9	
IQLCA-2026					X	X		Sqr. 2,0	X	Long	Sqr. 3,9	
IQMCA-2021					X	X		Sqr. 2,0	X	Medium	Counter Angle	
IMCA-0921	---	---	---	---	---	---	---	Hex. 0,9	X	Medium	Counter Angle	
IMCA-1621	---	---	---	---	---	---	---	Hex. 1,6	X	Medium	Counter Angle	
IMFCA-2021	---	---	---	---	---	---	---	Hex. 2,0	X	Medium	Counter Angle	
IMFCA-2521	---	---	---	---	---	---	---	Hex. 2,5	X	Medium	Counter Angle	
IQMCA-1321	---	---	---	---	---	---	---	Sqr. 1,3	X	Medium	Counter Angle	
IQMCA-3021	---	---	---	---	---	---	---	Sqr. 3,0	X	Medium	Counter Angle	
IFMCA-0321	---	---	---	---	---	---	---	Slot 0,4	X	Medium	Counter Angle	
ITMCA-1721	---	---	---	---	---	---	---	Torx 1,6	X	Medium	Counter Angle	

**APPLICATION**

The drivers for Emfils components are indicated for installation of implants and components for Emfils system components as shown in the table above.

The drivers for installing components from other systems are indicated for use in other implant systems as recommended by the component manufacturer.

**CONTRAINDICATIONS**

The bi-digital drivers do not present contraindications, provided they are used correctly for the purposes indicated.

**HANDLING**

Once sterilized, the installation driver should only be handled in a sterile environment by properly trained professionals and in appropriate costume at the time of surgery for the installation of dental implants.

All procedures must be performed by qualified professionals and the handling of the product is inherent in the training of the professional who will be responsible for using the instruments in the best possible way, respecting the Emfils indications for each instrument.

Check the correspondence of the prosthetic connection between the driver and the implant, or component that you want to install. Attach the driver interface to the torque wrench gear or counter-angle head, depending on the key chosen that best meets the user's need. Fit the tip of the driver to the implant or component and rotate clockwise until the planned torque is reached, always respecting the maximum torque recommended by Emfils for each product. For removal or disassembly, use the same attachment system, but the turn of the driver must be counterclockwise.

Important points in using the drivers:

- Avoid side loading.
- The drivers do not have torque control.
- Excessive force use can damage the product.

**SANITATION AND STERILIZATION**

The Emfils installation driver is reusable, provided non-sterile and should be sanitized and sterilized before and after use by following the steps mentioned in the link below:

[Novo Colosso's Surgical Kit Cleaning Manual](#)

**PRESENTATION**

The Emfils installation drivers are individually packaged in tyvek® surgical paper and transparent film display. The instrument undergoes a decontamination process that eliminates any type of dirtiness, but the sanitation process, described above, must be followed as soon as its first used.

**PRECAUTIONS**

- Do not use if the product packaging is broken;
- This product should be used sterile. Do not sterilize under dry heat (stove). Always use

steam and pressure autoclave;

- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- Check the correspondence of the prosthetic connection between the driver and the implant, or component that you want to install;
- Avoid side load on the switch;
- The drivers have no torque control. Only when used through the torque wrench will any measurement be possible;
- Maximum care is recommended when using the instruments so that they are not swallowed or aspirated by the patient;
- Before each procedure, check the conditions of the Emfils instruments, always respecting their useful life. Replace the instruments in case of damage, erasure, deformation and excessive wear.
- Before each procedure, make sure that the driver is properly fitted of the driver into the torque wrench gear or counter-angle head and the implant or component, otherwise it may occur the deformation, shearing, or even breakage of the driver, implant or component may occur in the application of torque;
- The use of excessive force can compromise the surgical act and damage the key, implant or component;

Always use the sequence of Emfils products. The use of instruments and / or prosthetic components from other manufacturers and systems does not ensure the perfect function and exempts Emfils from offering any product warranty. The drivers for installing components from other systems are indicated for use in other implant systems as recommended by the component manufacturer.

#### ADVERSE EFFECTS

Adverse effects will only occur if the choice of instrument or its handling is inadequate, which can cause damage to the surgical sequence of the implant to be installed.

#### STORAGE CONDITIONS

The product should be stored in a dry place and in the original packaging at room temperature and not expose to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

#### DISPOSAL OF MATERIAL

The professional must always evaluate the conditions of the instruments after each use. If the instrument is detected to be damaged, markings erased, deformed or excessive wear, must be disposed of in accordance with current regulations, complying with the environmental requirements and the criterion of potentially hazardous products, taking into account the different levels of contamination, physical risks

and selective collection.

#### LIFE CYCLE

The Emfils installation driver must be disposed when it loses its functionality, or the user detects excessive wear on the interface geometry.

#### SHELF LIFE

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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