IFU

(Instruction for use)

Saca Implante / Implant Remover / Saca Implante

PORTUGUÊS

ENGLISH

ESPAÑOL

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Favor contactar seu distribuidor autorizado.

This product has been designed to integrate clinical, mechanical and biological concepts, so, for best results it should only be used with original associated components according to Emfils instructions and recommendations.

All procedures must be performed by qualified professionals such as Implantodontists, Periodontists, Prosthodontists and Oral Maxillofacial Surgeons.

DESCRIPTION

The Emfils implant removers form a set of 3 pieces which have different lengths to better serve the user, which provides versatility to the system. At its active tapered tip, the product contains a 0.45 mm pitch left-hand thread that engages in the M2 internal threaded implant models. At its upper tip, the implant remover has a universal square fit of 3.9 mm, which connects the Emfils solid square driver (CBQ-4070), specially developed for use in implant removers. The raw material used in all of the drivers is tempered stainless steel, according to ASTM F 899.

The Emfils implant removers are shown in the table below:

Sales Code	Connection	External Diameter
TCM-2060	Morse Taper	3,0
TCE-2160	IH and EH	4,0

APPLICATION

The Emfils implant remover is a surgical instrument, indicated to be used manually in the removal of implants with internal thread M2, which already are osseointegrated. As the force for implant removal can be high, should always use the Emfils solid square driver (CBQ-4070). The implant remover is not indicated for removal of implants with an internal thread smaller than M2. It is recommended that the instrument be for single use only.

CONTRAINDICATIONS

The use of implant remover in situations of critical bone thickness is contraindicated. The use may cause damage to the bone.

HANDLING

Once sterilized, the implant remover should be used in compliance with all biosafety and surgical asepsis requirements.

All procedures must be performed by qualified professionals and the handling of the product is inherent in the training of the professional who will be responsible for using the instruments in the best possible way, respecting the Emfils indications for each instrument.

Attach the implant remover to the solid square driver (CBQ-4070), and attach it to the implant to be removed. By exerting digital pressure on the driver on the long axis of the implant, perform rotational movements counterclockwise until removal of the implant. As the wrench is massive, on each rotary movement, the wrench must be disconnected from the implant remover and replaced in a position of the mouth that facilitates a new movement.

Important points in using the instrument:

- Avoid side loading.

- The drivers do not have torque control.
- The initial force for interface fracture between osseointegrated implant and bone, is greater than during the procedure.

SANITATION

The implant removers are single use only, but are supplied non-sterile and should be sanitized and sterilized before and after use by following the steps mentioned in the link below:

[Novo Colosso's Surgical Kit Cleaning Manual](#)

PRESENTATION

The Emfils implant remover is individually packaged in tyvek® surgical paper and transparent film display. The instrument undergoes a process of decontamination that eliminates any type of dirtiness, however the process of hygiene, described above, must be followed. Need to be sterilized before using.

STERILIZATION

Supplied non-sterile. Indicated for single use.

STERILIZATION: is the procedure that aims at the total elimination of microorganisms, whether in vegetative or sporulated form.

Sterilization by Saturated Steam - Autoclaves

- In a conventional autoclave, the instrument should stand for 30 minutes at a temperature of 121 ° C when this temperature is reached.
- All surgical instruments and prosthetic components must be sterilized prior to use.
- Parts that have rubbers and plastics, for connection to another part, should not be autoclaved or sterilized being connected to each other. Example: Drill connected to Protractor.

PRECAUTIONS

- Do not use if the product packaging is broken;
- This product should be used sterile. Do not sterilize under dry heat (stove). Always use steam and pressure autoclave;
- Chemical sterilization is not recommended;
- Do not autoclave the product in its original packaging;
- A preliminary planning should be performed, evaluating the proximity of the implant to be removed with adjacent dental roots and noble anatomical structures, as this procedure can cause damage to these structures;
- Evaluate the condition of the implant to be removed, as it may be necessary to use trephine instead of the implant remover;
- Due to the reduced opening of the mouth in the posterior region, the use of the instrument may be difficult;
- Ensure that the pieces are not swallowed or aspirated by the person receiving the product (patient);
- Inadequate planning may compromise the performance of the procedure;
- Before each procedure, make sure that the parts are

perfectly inserted;

- Make sure you have all the necessary instruments to perform the surgery according to surgical planning;
- This product is for single use only;
- It is recommended to keep the implant remover axial along the axis of the implant to be removed. The inclination of the implant remover during torque application can result in damage and / or fracture of the implant or even the bone;
- It is the responsibility of the professional to use the Emfils products in accordance with the instructions for use.
- Reusing this product may cause:
 - *Adverse biological effects from remaining residues, whether microorganisms and / or substances from previous uses;*
 - *Changes in the physical, mechanical and chemical characteristics of macro or micro structural order, altering the original conditions of the product that may harm from its intended functionality;*
 - *The reuse of this product does not guarantee its safety and efficacy, and disclaims any warranty of related products.*
 - *Always use the sequence of Emfils products. The use of instruments and / or prosthetic components from other manufacturers and systems does not ensure the perfect function and exempts Emfils from offering any product warranty.*

ADVERSE EFFECTS

Adverse effects will only occur if the choice and use of the product is inadequate.

STORAGE CONDITIONS

The product should be stored in a dry place and in the original packaging at room temperature and not expose to direct sunlight. Incorrect storage may influence the original characteristics of the packaging and the product.

DISPOSAL OF MATERIAL

The implant remover is for single use only. The disposal of the material must comply with local regulations and environmental requirements, taking into account the different levels of contamination and selective collection.

LIFE CYCLE

Single use only.

DATE OF EXPIRATION

Written on the product label.

Not all products are available in all countries.

Please, contact the authorized distributor.

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